



READY. Set. Go.

Simply set in place and secure. No hydration required.

Use When:

- Antibacterial properties are needed to help prevent infection
- Absorption of moderate to heavy levels of exudate is required
- Compression therapy is needed
- Total contact casting is needed

Application:

- Simply place and secure; no hydration required
- Secondary dressing not required
- Tape or wrap required for fixation

Select Size and Apply



1 Select the appropriate dressing size to ensure the dressing will cover the entire wound.

Cover



2 Secure Hydrofera Blue READY with a fixation dressing (eg. adhesive tape).

Use with compression



3 Hydrofera Blue READY foam can be used in conjunction with compression wraps and total contact casting.

Wear Time



4 Dressing can be left in place up to 7 days, depending on the exudate level and clinical conditions of the wound.

Hydrofera Blue READY foam Ordering Information

Product No	Product Size	Quantity/Box
HBRS2520	6.4 cm x 6.4 cm (2.5" x 2.5")	10
HBRS4520	10.2 cm x 12.7 cm (4" x 5")	10
HBRS8820	20.3 cm x 20.3 cm (8" x 8")	10

For more information, see product instructions included with the product.

For product questions and samples please call our Customer Service Department at 1.800.263.7400 or your Hollister representative.

Manufactured for
Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60048 USA

Distributed in Canada by:
Hollister Limited
95 Mary Street
Aurora, Ontario L4G 1G3

www.hollister.com



The Hollister logo is a trademark of Hollister Incorporated. Hydrofera Blue Ready and the logo are trademarks of Hydrofera, LLC.

©2015 Hollister Limited.
922497-CA