



The peri-wound deserves silicone.

The wound deserves better.™

- TRIACT Technology helps create a moist, wound-healing interface
- TRIACT Technology helps prevent damage to newly formed tissue during dressing removal
- Silicone border offers virtually pain-free removal

Use for:

- Absorption of moderate to heavy exudate
- Protection of difficult wound-sites, peri-wound skin and bony prominences
- Simple application & easy removal
- Patient comfort

Indications for Use:

- TRIACT foam dressings with silicone border are indicated for moderate to heavily exuding acute and chronic wounds

For product questions or detailed clinical questions concerning our products in the US call 1.888.740.8999

Standard Dressing



Sacral Dressing



1 Prepare

- Cleanse the wound according to established procedures; ensure surrounding skin is dry prior to dressing application
- Choose dressing size so the central foam pad covers the entire wound

2 Place

- Remove one of the protective liners and position the dressing over the wound
- Remove the second protective liner Smooth down
- Apply gentle pressure to the dressing border to secure dressing
- Avoid stretching and pulling the dressing during application

3 Protect

- Dressing can be left in place for up to 7 days, depending on exudate level and clinical condition of the wound
- Waterproof backing will allow cleansing of surrounding areas per healthcare provider guidance

Ordering Information



TRIACT foam dressing with silicone border

Stock No	Product Size	Quantity/Box	HCPCS
550762	4" x 4" (10 cm x 10 cm)	10	A6212
550763	6" x 6" (15 cm x 15 cm)	10	A6212
550764	6" x 8" (15 cm x 20 cm)	10	A6213
550765	8" x 8" Sacral (20 cm x 20 cm)	10	A6213

Other foam dressings featuring TRIACT technology



Restore foam with TRIACT technology non-border

Stock No	Product Size	Quantity/Box	HCPCS
509380	2.5" x 2.5" (6 cm x 6 cm)	10	A6209
509381	4" x 4" (10 cm x 10 cm)	10	A6209
509382	6" x 6" (15 cm x 15 cm)	10	A6210
509384	4.7" x 7.5" (12 cm x 19 cm) Heel	10	A6210



Restore Lite foam with TRIACT technology non-border

Stock No	Product Size	Quantity/Box	HCPCS
509393	2.5" x 2.5" (6 cm x 6 cm)	10	A6209
509394	4" x 5" (10 cm x 12.7 cm)	10	A6210
509395	6" x 6" (15 cm x 15 cm)	10	A6210
509396	6" x 8" (15 cm x 20 cm)	10	A6210



Restore silver foam with TRIACT technology non-border

Stock No	Product Size	Quantity/Box	HCPCS
509345	4" x 4" (10 cm x 10 cm)	10	A6209
509346	6" x 8" (15 cm x 20 cm)	10	A6210

Caution: For dressings which contain silver, Federal (USA) law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

Prior to use, be sure to read the entire IFU package insert supplied with the product for device intended use, description, warnings, precautions, adverse events, and instructions for use.

The Hollister logo, The TRIACT logo, Restore, Restore Lite, and "The wound deserves better." are trademarks of Hollister Incorporated. Made in France. ©2015 Hollister Incorporated. 922512-1015



Manufactured for:
Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60048 USA.

Distributed in Canada by:
Distribué au Canada par:
Hollister Limited
95 Mary Street
Aurora, Ontario L4G 1G3
www.Hollister.com



TRIACT®